



PROTECH LEADED EYEWEAR INC,
DBA PROTECH MEDICAL

1360 North Killian Drive, Unit 2
Lake Park, FL 33403

Tel: (561) 627-9769
Toll-free: (888) 672-0923
Fax: (561) 627-0923
Email: sales@protechmed.com

APRONS & ACCESSORIES INSTRUCTIONS FOR USE (IFU)

1. GENERAL INFORMATION (PROTECH APRONS, AKA “PRODUCTS,” “GARMENTS,” “PPE”)

- Upon receiving your product, please inspect product for damages (holes, cuts, tears, rips, undone seams). Protech uses crush proof boxes when shipping PPE, but accidents do happen! Apron + thyroid should cover the wearer from throat to knee. This may vary based upon customization or special requests.
- For additional product details, please refer to the following: Product Inspection/Conformance form (in package) and product label (sewn on).

2. PROTECTION INFORMATION

EN 61331-1:2014: (EU) Protective devices against Diagnostic Medical X-ray radiation. (Determination of attenuation properties of material)

EN 61331-3:2014: (EU) Protective devices against diagnostic medical X-radiation Part 3: Protective clothing, Eyewear and Protective Patient shields.

ASTM 2547-06: (USA) Standard test method for determining the attenuation properties in a primary x-ray beam of materials used against radiation.

ASTM 2547-18: (USA) Standard test method for determining the attenuation properties in a primary x-ray beam of materials used against radiation.

3. MAINTENANCE: PROPER TRANSPORT, STORAGE, AND CLEANING TIPS

- Keep PPE out of direct sunlight and prolonged exposure to extreme heat. You may roll PPE and transport in a Protech duffel bag or box.
- Always store PPE on appropriate metal hanger. You may lay PPE down flat on the ground or a countertop. NEVER place objects on your PPE.
- Protech’s PPE Exterior and Interior fabrics (rip stop, nylon or prints, and vinyls) can be cleaned by using mild soap diluted with room temperature water. Clorox hydrogen peroxide in diluted form is permissible. **PDI Sani-Cloth® AF3 Germicidal Disposable Wipes are approved for use on Protech aprons.**
- Protech offers IntelliTrack, an apron inspection and tracking software, to help users during their scheduled apron inspections. These inspections are typically used to locate potential pinholes or cracks.

4. DISPOSAL AND REPAIRS | WARRANTY

- Users MUST dispose of PPE and accessories by complying with local, state, and federal or international regulations where applicable.
- For repairs, alterations, trade-ins, disposal, recovery and other services, please contact Protech or your local representative.
- Product is backed by a 2 year limited warranty against manufacturer defects.

5. EU TYPE EXAMINATION

Regulation (EU) 2016/425, Module B EU Type Examination Cert #: 523415/1

BTTG Testing & Certification Ltd.

Unit 14, Wheel Forge Way, Trafford Park, M17 1EH

B522 6 Manchester, UK

(Notified body #0338)

If you need additional Technical assistance, we are always here to help with lightning fast service and highly trained staff. Thank you for your loyalty and support.